

Learn More About
iScreen OFD Employee Drug Testing Kits



EmployeeDrugTestingAce.com
Trusted Proven **Secure**

www.DrugTestingAce.com

www.EDTAce.com



inverness medical innovations

Drugs of Abuse Business Unit
iScreen OFD Training



inverness
medical
innovations

The iScreen OFD

- ◆ iScreen OFD Oral Fluid Test Device
 - Rapid on-site drug screening test device for the detection of drugs in oral fluid
 - Collect anywhere, anytime
 - Results within minutes
- ◆ iScreen OFD is available with up to 6 different drugs
 - AMP: Amphetamines
 - mAMP: Methamphetamine/MDMA
 - COC: Cocaine
 - OPI: Opiates
 - PCP: Phencyclidine
 - THC: Marijuana
- ◆ Gather all necessary testing supplies
 - iScreen OFD device in foil pouch
 - Timer or stop watch



The Process: Performing the Drug Screen


◆ Give Test Device to Donor

Note: Test Administrator should not handle the test device once handed to the donor, until the security seal has been applied and Chain of Custody has been established.

- Instruct donor to open device
- Have donor remove plastic sponge protector and discard

◆ Set aside blue device cap

Verify that plastic port seal is on cap
If not, remove from pouch and insert into cap

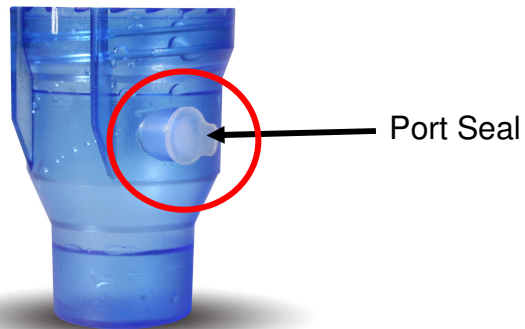

Sponge Protector



The Process: Sample Collection

- ◆ Instruct donor on proper procedure for saliva collection
 - Ensure that the donor has not placed anything in mouth for at least **10 minutes** prior to collection. Including food, drink, tobacco products, gum, candy, etc.
 - Actively swab inside of mouth and top of tongue for **3 minutes**
 - Allow saliva to saturate sponge
 - Sponge should soften completely



The Process: Initiating Test



- ◆ Donor secures blue cap onto the test device
 - The test device should be held vertically and inserted into the blue cap, tightening until the red arrow on the test device is aligned with one of the white marks on the cap.
 - Once the cap is in place, the saliva will wick up the test strips – you will see a pink migration indicating proper saliva flow.
 - If either of the test strips fails to wick, have the donor loosen the blue cap $\frac{1}{2}$ to $\frac{3}{4}$ of a turn and then re-tighten. This allows the air pressure within the device to equalize and the sample to flow properly.

Failure to wick properly can be caused by insufficient sample volume, extremely viscous saliva, or an air bubble blocking the flow.

The Process: Securing the Device

◆ Test Completion

- Instruct donor to initial and date the silver security seal included in test kit box
 - Donor will place this security seal over device

KEEP THE TEST DEVICE IN VIEW OF BOTH THE DONOR AND ADMINISTRATOR UNTIL DEVICE IS SEALED WITH SECURITY SEAL

ADMINISTRATOR SHOULD NOT HANDLE DEVICE UNTIL SECURITY SEAL IS IN PLACE



Results: Interpretation and Documentation

- ◆ Interpret the test results at 10 minutes
- ◆ Results are stable for up to one (1) hour
- ◆ Each test strip in the device includes an internal procedural Control (C) ensuring proper device function.
- ◆ Control lines should form next to the C on each of the test strips.
- ◆ Test interpretation:
 - A line next to the drug name indicates a negative result for that drug.
 - The absence of a line next to the drug name indicates a presumptive positive for that drug.

It is recommended that drug screen results should not be given to donor.

Results: Interpretation and Documentation

Invalid Test Result

- ◆ Each test strip within the device includes an internal procedural Control (C) ensuring proper device function.
- ◆ Control lines should form on both test strips, indicating the test functioned properly.
- ◆ The absence of a Control line in either of the test strips indicates the test is **INVALID**.
- ◆ Discard the test device.
- ◆ Perform another test with a new sample and a new device.



Results: Interpretation and Documentation

Negative Test Result


- ◆ The appearance of a line next to every drug name and **BOTH** control areas
- ◆ Discard test device
- ◆ **Any indication of a line regardless of intensity is correctly interpreted as a negative test result.**



Results: Interpretation and Documentation

Presumptive Positive Test Result

- ◆ The absence of a line next to the drug name indicates a presumptive positive for that drug.
- ◆ Example shows: presumptive positive test result for THC

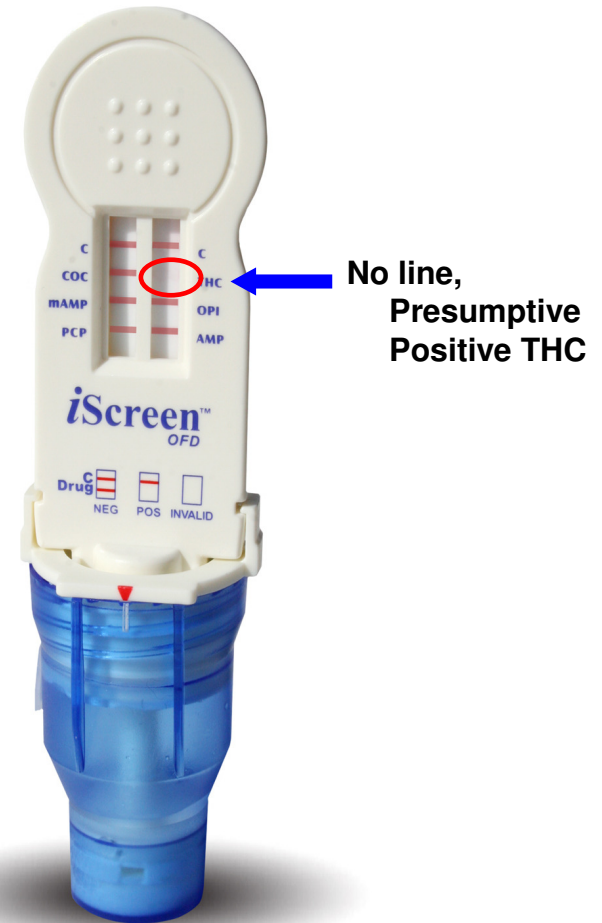

Brought to You by Employee Drug Testing Ace

Buying discounted, accurate, high quality employment drug testing kits and services is affordable, fast, easy, safe and secure with **Employee Drug Testing Ace**. You can buy employment, pre-employment and employee drug and alcohol testing kits, services and supplies tailored for employers, TPAs, nursing homes, hospitals, manufacturing, moving companies, Department of Transportation DOT, local and Federal Government agencies, auto dealerships, construction, DOD, the work place, employment agencies and more.

Our easy to use [employee drug testing](#) kits are DOT Approved, CLIA Waived, FDA 510k Cleared and SAMHSA Compliant. We provide both onsite employment drug screening and offsite employee drug screening services along with SAMHSA certified confirmation and medical review officer (MRO) services. All [employment drug screening](#) kits are on sale, discounted, and have [bulk / volume discounts](#).

- **Pay No Taxes in the USA.**, except California.
- **100% Free UPS Standard Shipping** in the Continental U.S.A. for *All* Products.
- **100% Free FedEx Overnight Shipping** in the Continental U.S.A. for *All* Specimens sent in as part of our SAMHSA Certified Confirmation Drug Testing Service.
- Onsite Third Party Administered (TPA) Employee Drug and Alcohol Collection Services.
- Over 12,000 Offsite Employee Drug and Alcohol Testing Locations Nationwide.
- The most complete [Employment Drug Testing Resource Center](#) including: information, official documents, work place drug testing information, forms, work accident info, charts, videos, post accident drug testing info, training materials, contact lists, definitions and more!

We sell affordable and guaranteed [employee drug screening](#) kits and services available at the best prices. Our prices may be cheap but our drug testing kits are of the highest quality and accuracy. If you are in the market for discounted high quality pre-employment, employee, or [employment drug testing](#) and screening kits and services, visit us today. You will be happy you did!