



Learn More About
OrAlert Employment Drug Testing Kits

 **EmployeeDrugTestingAce.com**
Trusted Proven **Secure**

www.DrugTestingAce.com www.EDTAce.com

OrAlert™ Drug Screen

Product Benefits

Test Device

- Donor collects his or her own specimen
- Donor expresses specimen into chamber
- Test administrator has minimal chance of contact with specimen

No Additional Equipment Required


- Product design allows for visual interpretation
- Test administrator is not required to wear gloves.

Test Admin. & Lab Concerns Addressed

- Minimal chance of contact or contamination of specimen
- Dedicated specimen retention chamber




Description of Items

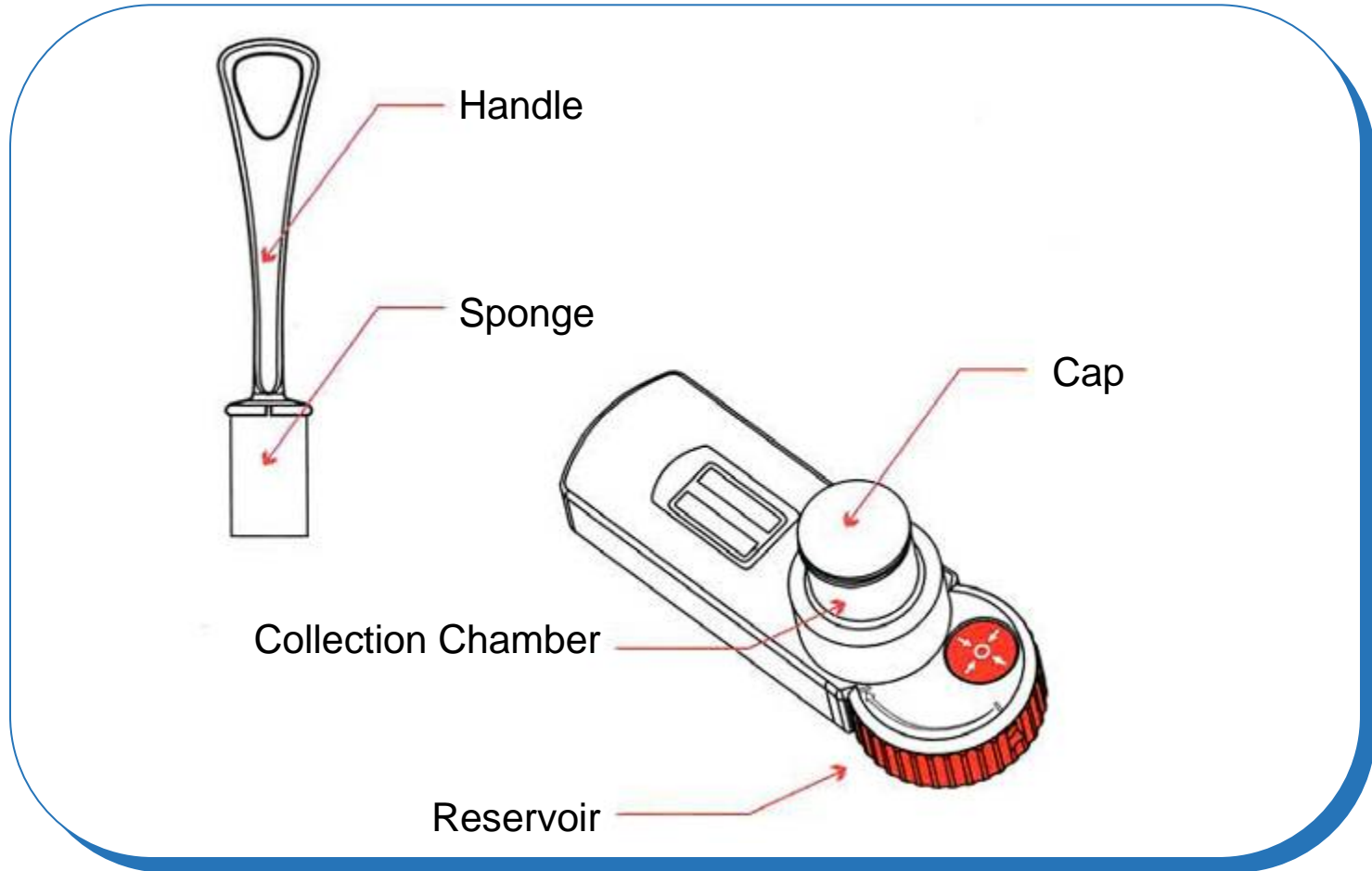


Step 1.

- Remove cap. Make sure donor hasn't eaten or had any liquids to drink within the past ten (10) minutes.
- Ensure reservoir is locked in testing position by turning counterclockwise as far as possible.

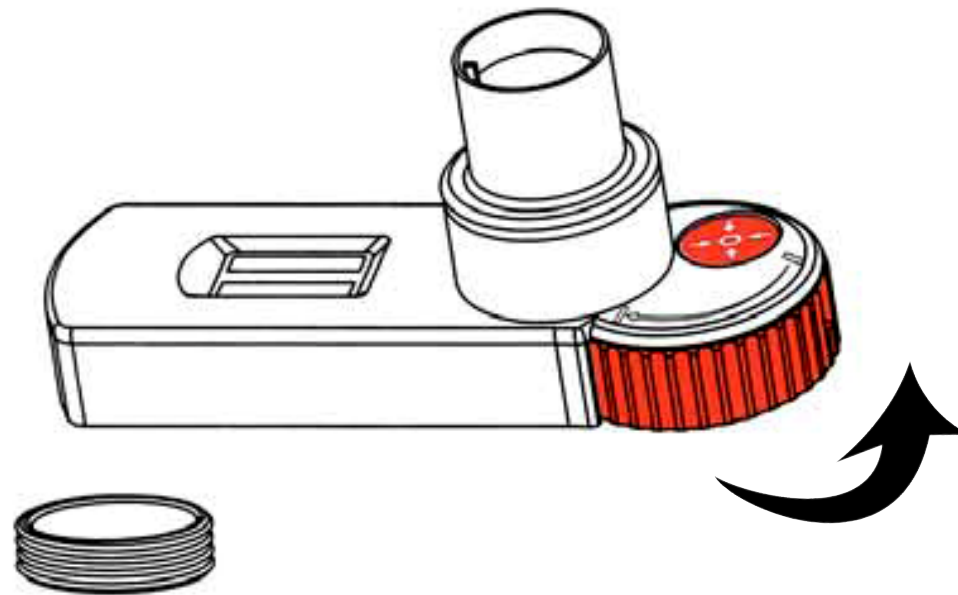


Step 2.

- Insert sponge end of collector into mouth.

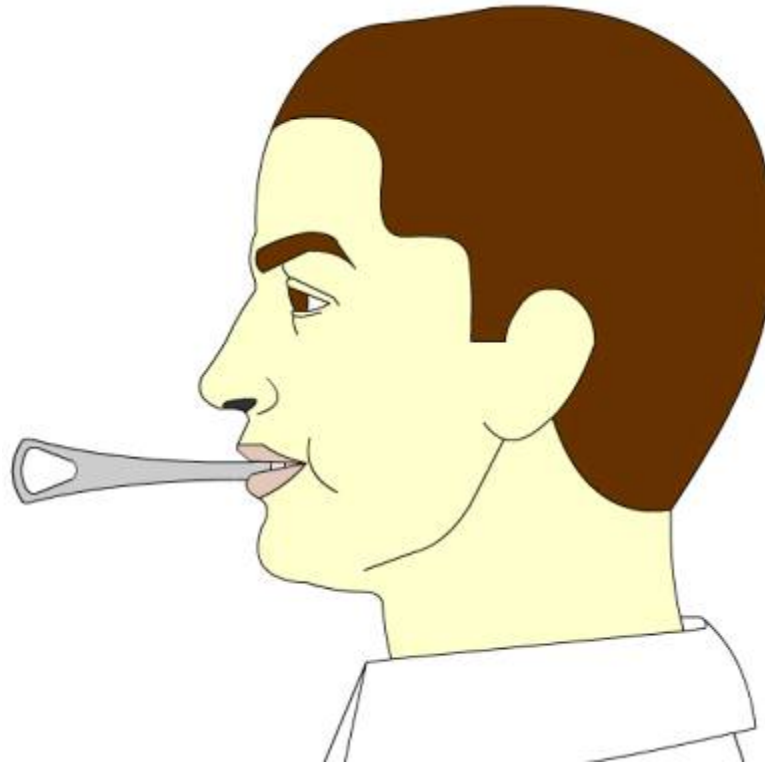


Step 2.

- Actively swab the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press the sponge between the tongue and teeth to ensure complete saturation.

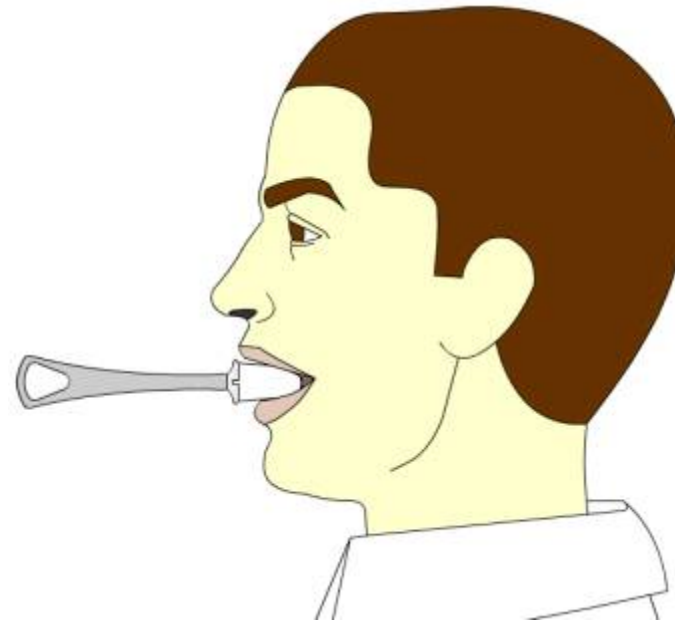


Step 2.

- The sponge is saturated when no hard spots can be detected. Collect for a total of (3) minutes before removing sponge.

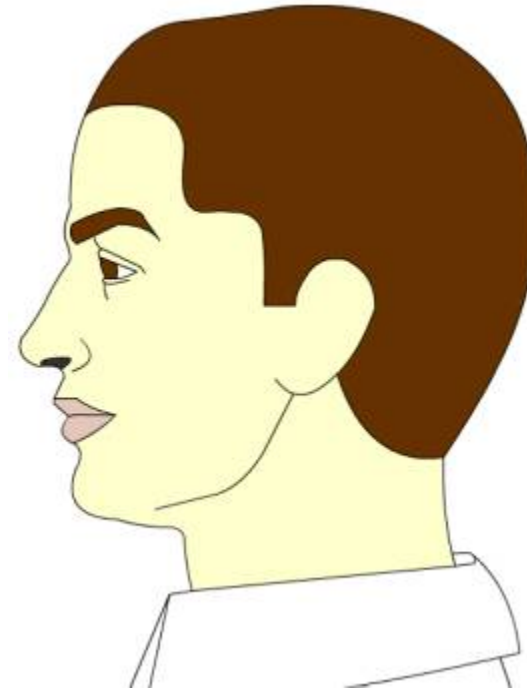


Step 3.

- Insert collector into device as shown.
- Push collector down into chamber.
- Turn collector clockwise to lock into chamber.

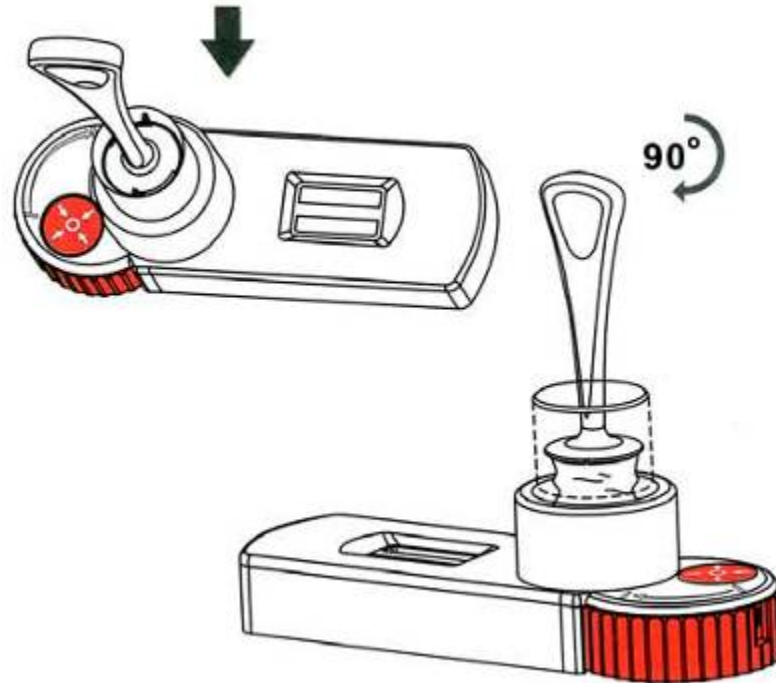

Step 4.


- Wait 10 minutes, remove collector from device and read results.

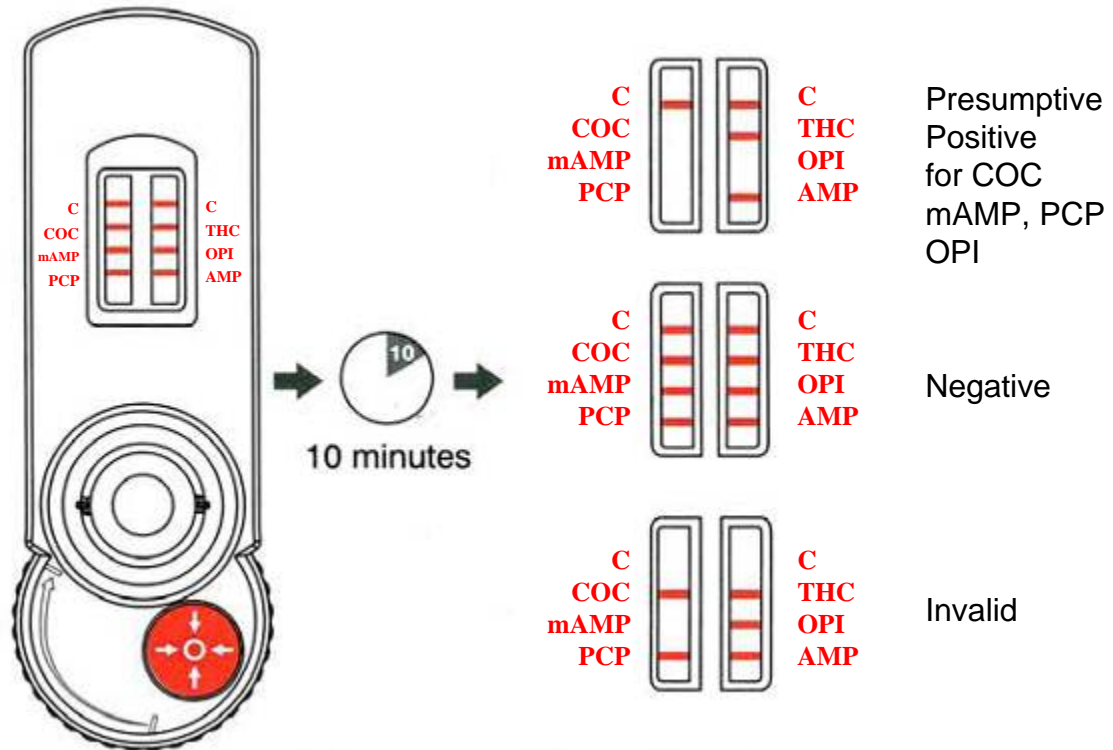

All positive results are presumptive and should be confirmed by an alternate method (e.g. GC/MS or GC/MS/MS).

Step 5.

- Secure cap back onto collection chamber.
- Turn reservoir clockwise as far as possible to lock.
- If results are presumptive positive, secure reservoir with tamper evident seal and send to laboratory for confirmation.

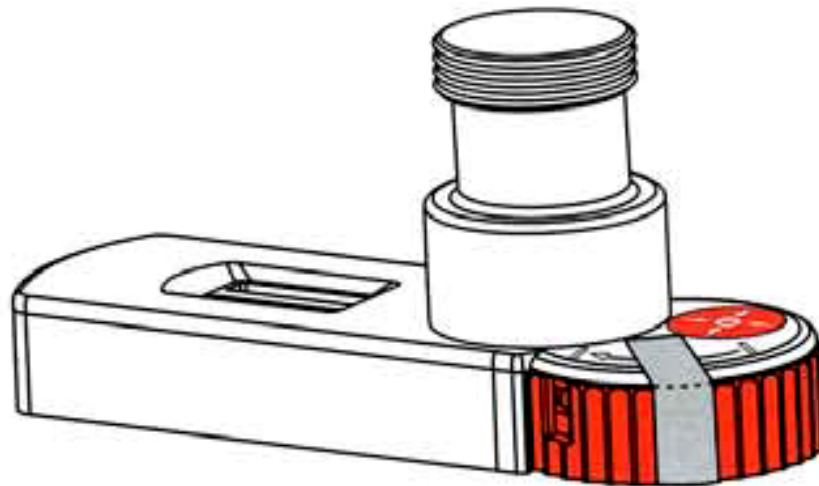



Brought to You by Employee Drug Testing Ace

Buying discounted, accurate, high quality employment drug testing kits and services is affordable, fast, easy, safe and secure with **Employee Drug Testing Ace**. You can buy employment, pre-employment and employee drug and alcohol testing kits, services and supplies tailored for employers, TPAs, nursing homes, hospitals, manufacturing, moving companies, Department of Transportation DOT, local and Federal Government agencies, auto dealerships, construction, DOD, the work place, employment agencies and more.

Our easy to use [employee drug testing](#) kits are DOT Approved, CLIA Waived, FDA 510k Cleared and SAMHSA Compliant. We provide both onsite employment drug screening and offsite employee drug screening services along with SAMHSA certified confirmation and medical review officer (MRO) services. All [employment drug screening](#) kits are on sale, discounted, and have [bulk / volume discounts](#).

- **Pay No Taxes in the USA.**, except California.
- **100% Free UPS Standard Shipping** in the Continental U.S.A. for *All* Products.
- **100% Free FedEx Overnight Shipping** in the Continental U.S.A. for *All* Specimens sent in as part of our SAMHSA Certified Confirmation Drug Testing Service.
- Onsite Third Party Administered (TPA) Employee Drug and Alcohol Collection Services.
- Over 12,000 Offsite Employee Drug and Alcohol Testing Locations Nationwide.
- The most complete [Employment Drug Testing Resource Center](#) including: information, official documents, work place drug testing information, forms, work accident info, charts, videos, post accident drug testing info, training materials, contact lists, definitions and more!

We sell affordable and guaranteed [employee drug screening](#) kits and services available at the best prices. Our prices may be cheap but our drug testing kits are of the highest quality and accuracy. If you are in the market for discounted high quality pre-employment, employee, or [employment drug testing](#) and screening kits and services, visit us today. You will be happy you did!