

# **iCup**<sup>®</sup> Drug Screen



## **iCup**<sup>®</sup> Drug Screen

*Corporate Drug Testing Made Simple.*

- Available in 3-13 Drug Configurations
- Specimen Validity Testing Up To 6 Parameters
- Room Temperature Storage
- Built-in Validity Test
- 24-month Shelf Life
- Fast Results

*\* iCup AD available on some configurations.*

**Innovative Drug Screening Solutions**  
**Quality • Choices • Support**

Learn More About  
**iCup Employment Drug Screening Kits**



**EmployeeDrugTestingAce.com**  
Trusted Proven Secure


[www.DrugTestingAce.com](http://www.DrugTestingAce.com)

[www.EDTAce.com](http://www.EDTAce.com)

C	C	C	C	C
1	1	1	1	1
2	2	2	2	2
3	3	3	3	3
1-AMP	1-mAMP	1-BZO	1-OXY	
2-COC	2-OPI	2-TCA	2-PPY	
3-THC	3-PCP	3-BAR		

Instant Technologies, Inc.



an  inverness medical company

# iCup<sup>®</sup> A.D. Drug Screen

## iCup<sup>®</sup> A.D. Drug Screen Corporate Drug Testing Made Simple.

Detect the presence of drugs and adulterants in minutes. It's drug testing efficiency you can count on.

### Integrated testing solution:

- iCup<sup>®</sup> AD with temperature strip
- Security Seal
- Two-part result form available
- Photocopy Template




iCup<sup>®</sup> AD 10-Panel shown as a negative result.

Drug Name:	Abbreviation:	Cutoff Levels:
Amphetamine.....	AMP.....	1000ng/ml
Barbiturates.....	BAR.....	300ng/ml
Benzodiazepines.....	BZO.....	300ng/ml
Buprenorphine.....	BUP.....	10ng/mL
Cocaine.....	COC.....	300ng/ml
Ecstasy.....	MDMA.....	500ng/ml
Marijuana.....	THC.....	50ng/ml
Methadone.....	MTD.....	300ng/ml
Methamphetamine.....	mAMP.....	1000ng/ml
Morphine.....	MOR.....	300ng/ml
Opiates.....	OPI.....	2000ng/ml
Oxycodone.....	OXY.....	100ng/ml
Phencyclidine.....	PCP.....	25ng/ml
Propoxyphene.....	PPX.....	300ng/ml
Tricyclic Antidepressants.....	TCA.....	1000ng/ml

## Instructions


### 1 Collect Specimen

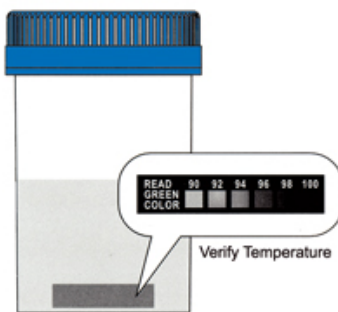


### 2 Peel Back Label



### 3 Read the Results

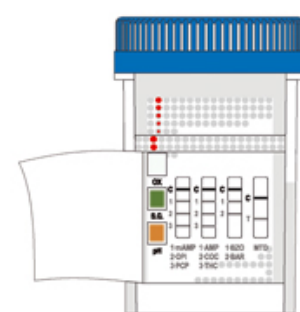

Read results at five (5) minutes.  
Results stable up to sixty (60) minutes.

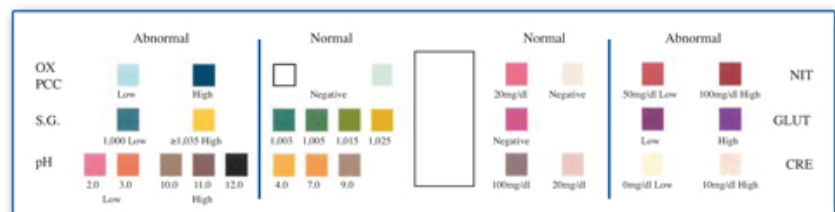

SAMPLE ONLY - Color chart NOT Exact