

Fast and convenient oral fluid drug detection for employment and insurance testing.




For in vitro diagnostic use. For use in employment and insurance testing.

T-Cube® Oral Fluid Drug Test


With numerous oral fluid drug test devices on the market, it is important that you select the one best suited for your particular setting and needs. The **T-Cube®** oral fluid drug test is intended for employment and insurance testing, and offers fast and convenient oral fluid detection for multiple drugs assays. Oral fluid drug screening simplifies collection processes and protects against specimen tampering—allowing you to make informed decisions about substance abuse.

- 6-10 drug configurations available for a fully customized oral fluid testing program
- Saliva saturation indicator ensures sufficient specimen volume is collected
- Eliminates need for gender-specific observation and limits adulteration and tampering
- Drug test results in 5 minutes

Easy to Use

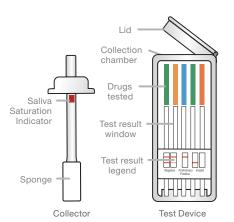


Put collector sponge on tongue or near cheek to collect oral fluid for about 3 minutes (until saturation indicator strip appears red).

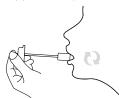

2

Open the test cube and place the saturated collector inside. Press firmly until it reaches the bottom. Close the test cube lid. Keep cube upright on a flat surface.

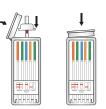

3

Interpret drug test results at 5 minutes. Do not read after 5 minutes.



PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations, and warnings. For in vitro diagnostic use. For use in employment and insurance testing. The assay provides a qualitative, preliminary test result. A more specific analytical method must be used in order to obtain a confirmed result. Gas chromatography-mass spectrometry (GC-MS) or liquid chromatography-tandem mass spectrometry (LC-MS/MS) are preferred confirmatory methods. Professional judgment should be applied to any drug test result, particularly when preliminary results are positive.

Ordering Information

Item #	T-Cube® Oral Fluid Drug Test	Packaging
TCube-6MB	6 Panel: AMP50, BZO30, COC20, MET50, OPI40, THC25	25 tests per box
TCube-60	6 Panel: AMP50, COC20, MET50, OPI40, OXY20, THC25	25 tests per box
TCube-8BUP	8 Panel: AMP50, BUP5, BZO30, COC20, MET50, OPI40, OXY20, THC25	25 tests per box
TCube-9BUP	9 Panel: AMP50, BUP5, BZO30, COC20, MET50, MTD30, OPI40, OXY20, THC25	25 tests per box
TCube-10BUP	10 Panel: AMP50, BAR60, BUP5, BZO30, COC20, MDMA100, MET50, OPI40, OXY20, THC25	25 tests per box
TCube-10MO	10 Panel: AMP50, BAR60, BZO30, COC20, MDMA100, MET50, MTD30, OPI40, OXY20, THC25	25 tests per box

Drug Tests: Amphetamine (AMP), Barbiturates (BAR), Buprenorphine (BUP), Benzodiazepines (BZO), Cocaine (COC), Methamphetamine (MET), Ecstasy (MDMA), Methadone (MTD), Opiates (OPI), Oxycodone (OXY), Marijuana (THC) 11-nor-Δ9-THC-9-COOH.

